



Precision Xceed Pro with TrueID to support patient identification



**Precision
Xceed Pro**

Integrated Safeguards for
Real-World Clinical Accuracy

Precision Xceed Pro is so simple to operate – reduces the potential for error

Ergonomic design facilitates single-handed operation


- The Precision Xceed Pro meter can be held comfortably in either hand



- Tough casing – In tests, Precision Xceed Pro has been dropped from heights of 2m without damage¹

TrueID simplifies data entry and supports positive patient identification


- Scanning the patient's wrist band, the operator's badge and the test strip's bar code speeds data entry and reduces risk of data entry errors
- The soft alphanumeric keypad is easy to use for manual entry and confirmation
- The monitor displays patient name, date of birth and gender to help patient identification

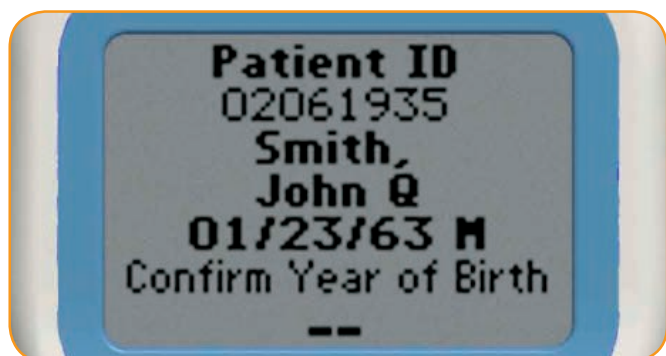

1. Data on file, Abbott Laboratories

Dual-fill strip simplifies blood application

- Test only starts when fill trigger confirms sufficient blood volume is present
- The meter beeps audibly when sufficient blood is applied, and the 20 second countdown begins

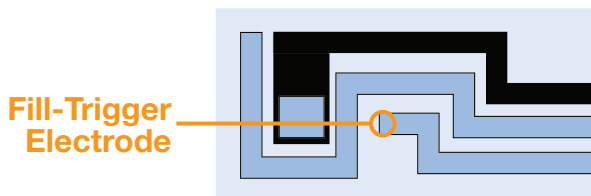


Testing is straightforward with Precision Xceed Pro

- Scan patient's wrist band, operator's badge and test strip's bar code
- Tear open foil and insert strip into port
- Apply blood drop to strip. When adequate blood is applied, countdown begins



Straightforward testing.
Results you can trust

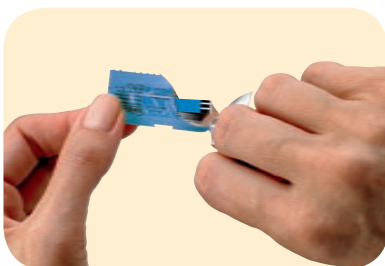



**Precision
Xceed Pro**

Precision Xceed Pro

- ✓ Simple to operate
- ✓ Ergonomic design
- ✓ Automatic calibration via the strip bar-code scan
- ✓ Backlit display
- ✓ Scanning data entry
- ✓ Patient details – helps identification
- ✓ TrueID – secure data

Precision Xceed Pro test strips

- ✓ Individually wrapped
- ✓ Safeguards prevent short sampling
- ✓ Audible signal when sufficient sample applied
- ✓ Glucose-specific chemistry to reduce interferents
- ✓ End-fill or top-fill options

Precision Xceed Pro with TrueID delivers the results you need



Abbott Diabetes Care, Abbott House, Vanwall Business Park,
Vanwall Road, Maidenhead, Berkshire, SL6 4UD.
www.abbottdiabetescare.co.uk

Abbott Laboratories, Abbott Diabetes Care, 4051 Kingswood Drive,
Citywest Business Campus, Saggart, Dublin 24
www.abbottdiabetescare.ie

Precision Xceed Pro and TrueID are trademarks of the Abbott Group of companies in various jurisdictions.
002PXP200711

